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**EN** Olej do dissolving gutta-percha

**BEFORE USE PLEASE READ THE FOLLOWING INSTRUCTIONS  
CAREFULLY  
TO BE USED BY DENTIST ONLY**

**COMPOSITION:** eucalyptus oil 100%

**INTENDED USE**

EUCALYPTOL is intended for use during repeated endodontic treatments to remove gutta-percha from the root canal.

**INSTRUCTIONS FOR USE**

Before the treatment, immerse the attached pipette into ethyl alcohol.

1. Using the pipette, insert 1-2 drops of liquid into the root canal.

2. Remove gutta-percha.

3. Use another dose of preparation if necessary.

4. If gutta-percha is still dry or crumbly, apply EUCALYPTOL again.

5. Repeat as needed.

The product is reusable while the pipette is disposable. Multiple use of the pipette may cause the risk of re-infection.

**CONTRAINDICATIONS**

Do not use for patients who are hypersensitive to the product components.

**PRECAUTIONS**

It can cause allergic skin reaction. Flammable liquid and vapour. Use rubber-dam, gloves, protective glasses and protective clothing while working with the product.

Any serious incident involving the product should be reported to the manufacturer and the competent authority of the State in which the user or patient is domiciled.

**STORAGE**

Store in original packaging below 25°C.

Keep out of reach of children.

Expiry date is placed on the main container. Once container is open, the expiry date does not change if the container is securely closed after each use.

**HANDLING THE PRODUCT CONTAINER**

Empty containers should be disposed of responsibly or returned to the manufacturer.

**PACKAGING**

Bottle with 10 ml of preparation, application pipettes (x3).

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